

Prior to attempting to update your Bionet BM3Vet or BM3Vet plus monitor you need to determine which update file you need.

1. Run a test strip to determine the current version of the installed software. Press the Print button and let the strip run for 10 to 15 seconds. You will see on the bottom of the strip:
"BM3Vet/BM3 Vet Plus Version X.XXXXXX.XX>XX Bionet Co., LTD" -- Note: The Xs will be the actual version number and the text may vary slightly from model to model.

2. Use the following table to determine the update file you need:

BM3Vet	
Current Version	Update File
1.05 to 1.11	BM3 Vet 1.13BVA.bin
2.02	430_DIGI_203V_100.100.bin
4.00	BM3 Vet_4.00BVB.bin
BM3Vet Plus	
Current Version	Update File
1.05 to 1.12	BM3 Vet Plus_1.13BVA.bin
2.02	440_DIGI_202V_100_100.bin
4.00	BM3 Vet Plus_4.00BVB.bin