



Sales Notes

B-Link v1.0

April 2007



What's B-Link?

B-Link is a software program that enables you to interface Bionet veterinary monitors, including the **BM3Vet**, **BM3Vet Plus** and **BM5Vet**, directly to Windows based computers. The patient data can be saved as a formatted text file or can be pasted directly into the electronic medical record of the veterinary practice management software you already use!




Why B-Link?

Traditionally, if a veterinarian wanted to import the results from their patient monitor to their practice management software they would have to contact the company that provides the software and hope they would write an interface.

Bionet America has eliminated that step. Our B-Link product will be compatible with all Windows based practice management software.




Connection Methods

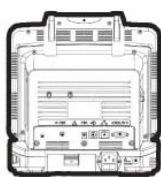

Crossover Cable



Computer to Computer

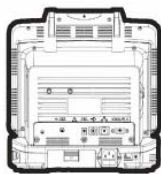

Ethernet LAN Cable



Standard Networking



Screen & Features

① Basic demographic information entered here

Client / Pet information

Client: JOHN DOE Pet: SPARKY Sex: MN
 ID: 3410 Species: CANINE DOB: 03/04/2002
 Phone: 555-555-5555 Breed: SPITZ Weight: 345LBS
 Misc: Over Weight

Check list: BM5vet.LIF 30 Seconds

Date and Time	HR	SPO2-%	SPO2-R	RESP	NIBP-S	NIBP-M	NIBP-D	TEMP1
3/27/2007 5:02:29 PM	120	98	119	30	145	100	85	100.94
3/27/2007 5:02:49 PM	120	98	119	30	145	100	85	100.94
3/27/2007 5:03:19 PM	120	98	119	30	145	100	85	100.94
3/27/2007 5:03:49 PM	120	98	119	30	145	100	85	100.94
3/27/2007 5:04:19 PM	120	98	119	30	145	100	85	100.94
3/27/2007 5:04:49 PM	120	98	119	30	145	100	85	100.94
3/27/2007 5:05:19 PM	120	98	119	30	145	100	85	100.94
3/27/2007 5:05:49 PM	120	98	119	30	145	100	85	100.94
3/27/2007 5:06:19 PM	120	98	119	30	145	100	85	100.94
3/27/2007 5:06:49 PM	120	98	119	30	145	100	85	100.94

② Parameter selection and set transfer interval

③ Paste results directly into ANY Windows based software

④ Data can be saved in formatted text file(csv file)


Paste into target program i.e. Notepad

Copy to Clipboard


Copy to Clipboard

Can be pasted into ANY Windows based software

Saved as formatted text

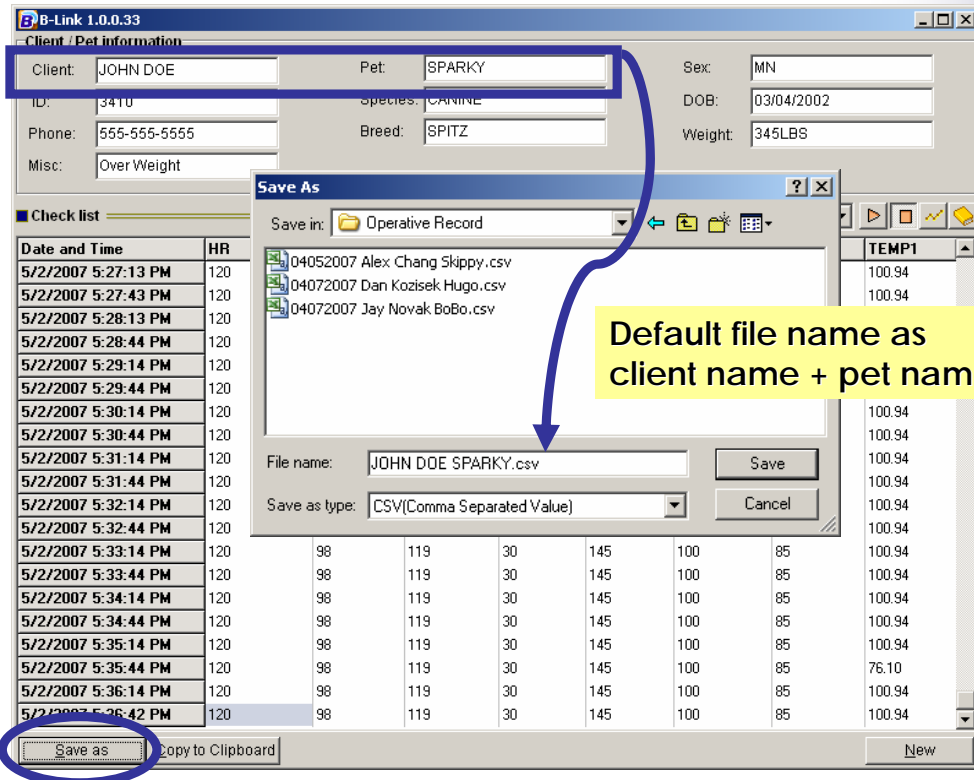

Saved as formatted text(cont'd)

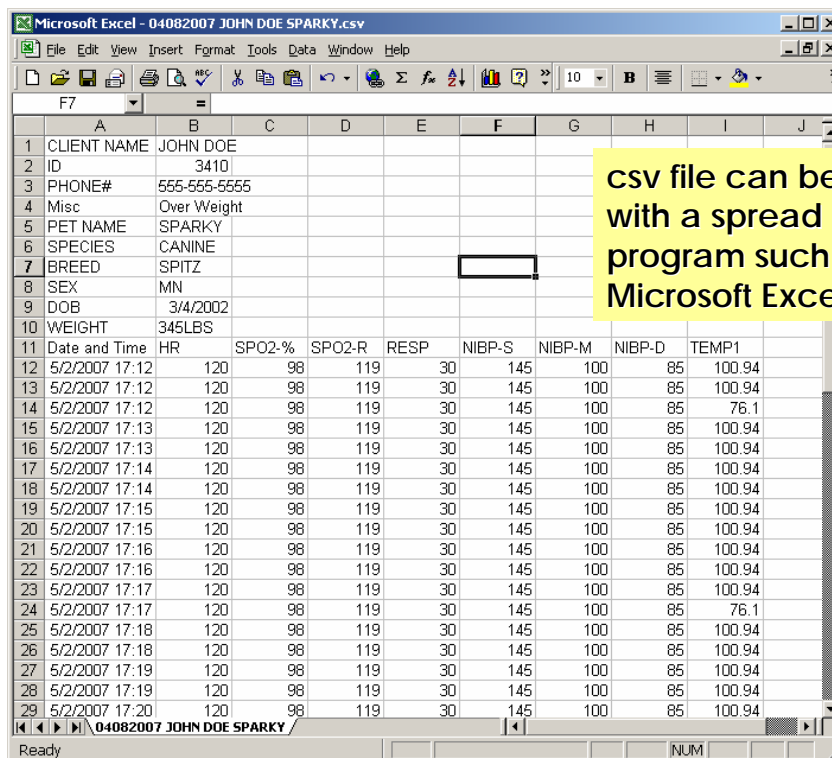

Data transfer by unit

Parameter	BM3Vet	BM3Vet Plus	BM5Vet	BM5Vet-CS(EtCO2)
HR	●	●	●	●
SPO2-%	●	●	●	●
SPO2-R	●	●	●	●
RESP	●	●	●	●
NIBP-S	●	●	●	●
NIBP-M	●	●	●	●
NIBP-D	●	●	●	●
TEMP1*	●	●	●	●
TEMP2*			●	●
IBP1-S*			●	●
IBP1-M*			●	●
IBP1-D*			●	●
IBP1-P*			●	●
IBP2-S*			●	●
IBP2-M*			●	●
IBP2-D*			●	●
IBP2-P*			●	●
ETCO2				●
ETCO2-R				●
FIO2				●



Customers Benefit

A. Efficiency of Operation

No more having your techs transcribing vital signs into your medical record. Eliminates transcription errors and frees your techs to be techs.

B. Convenience of Record Keeping

Store all data in digital format. No need to keep track of a printed strip in your paper chart.



B-Link is compatible with



IDEXX Cornerstone®
Practice Management Software



and with many more Windows based
veterinary practice management software!



B-Link Availability



Monitor	Screen Size	ECG	SpO2	NIBP	Resp	Temp	EtCO2	IBP	Arrhythmia Analysis	B-Link
BM3Vet	5.7"	x	x	x	x	opt	n/a	n/a	n/a	opt
BM3Vet Plus	8.4"	x	x	x	x	opt	n/a	n/a	n/a	opt
BM5Vet	10.4"	x	x	x	x	opt dual	opt	opt	x	x

x=standard, opt=optional, n/a=not applicable





Thank you!

Product inquiry:
Bionet America, Inc.

Tel 714-734-1760
Fax 714-734-1761

sales@bionetus.com
support@bionetus.com

