


## Bionet Veterinary NIBP Guideline

The most important factors in getting a successful and accurate blood pressure are proper cuff size selection and placement.


The cuffs are all graduated so you can easily find the correct sized cuff to use. There is an index point marked on the leading edge of the cuff. When wrapped around the limb, this index point must be within the graduated scale.



Child cuff showing Index Line and Graduated Scale

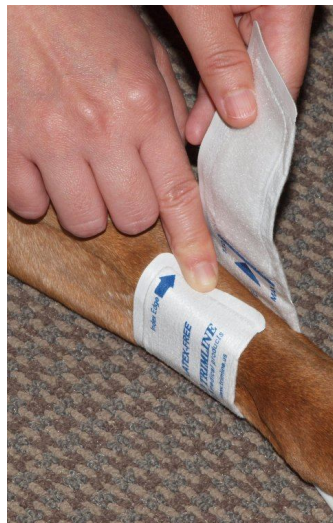

Cuff is too small. Index edge won't fall within scale.

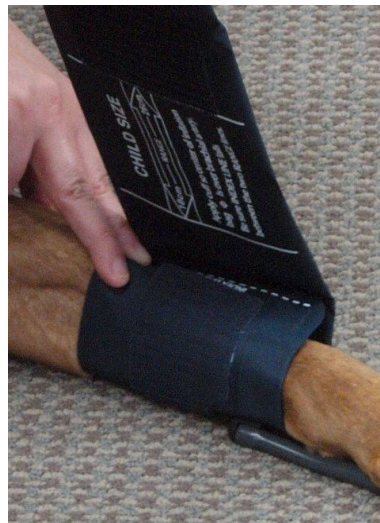

Cuff is too large. Index line won't fall within scale.



Cuff is correct size.

The hose should line up with the vessel that you are trying to measure. When using a leg, this is on the underside (backside) of the leg.



**Incorrect placement**



**Correct placement**

You can use any leg or the base of the tail to get a blood pressure. On smaller cats, you may have to place the cuff above the hock.

Note: You may have to shave the area if the pet has thick fur and/or is particularly small.

Make sure the cuff size setting in the patient monitor is set correctly. Cuffs #1 and #2 are SMALL. Cuffs #3 and #4 are MEDIUM and all other cuffs are LARGE.

MAIN MENU	ALARM	CUFF SIZE: MEDIUM	> LARGE MEDIUM SMALL
PREV MENU	UNIT SELECT: mmHg		

For any questions, please contact Customer Service at 1.877.924.6638 or [support@bionetus.com](mailto:support@bionetus.com).