

Document Type: Customer Service
Document No.: BA-SVC-09A

Bionet Veterinary Monitor Connector Cheat Sheet (BM3Vet / BM3Vet Plus / BM3Vet Next / BM5Vet)

: How to tell the difference between
an old style connector and a new one

Revised: February 25, 2009



Customer Service
Bionet America, Inc.
2691 Dow Ave. Ste B
Tustin, CA 92780
Toll Free: 1.877.924.6638
Fax: 714.734.1761
support@bionetus.com
www.bionetus.com

A. SpO2 Extension Cable


We have three variations on SpO2 Extension Cable:

1. Old Style

Item Code: B-SPCBL-O, Item Description: SpO2 Extension Cable - Old



Connector on Cable

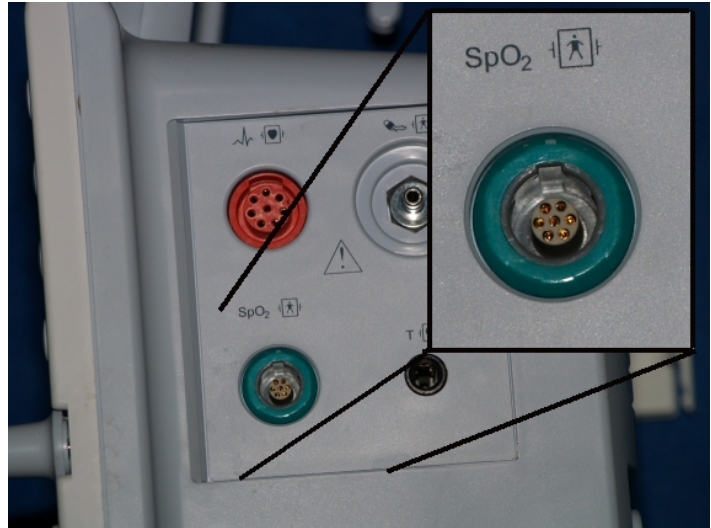


Connector on Monitor

2. New Style

Item Code: B-SPCBL-N, Item Description: SpO2 Extension Cable - New




Connector on Cable

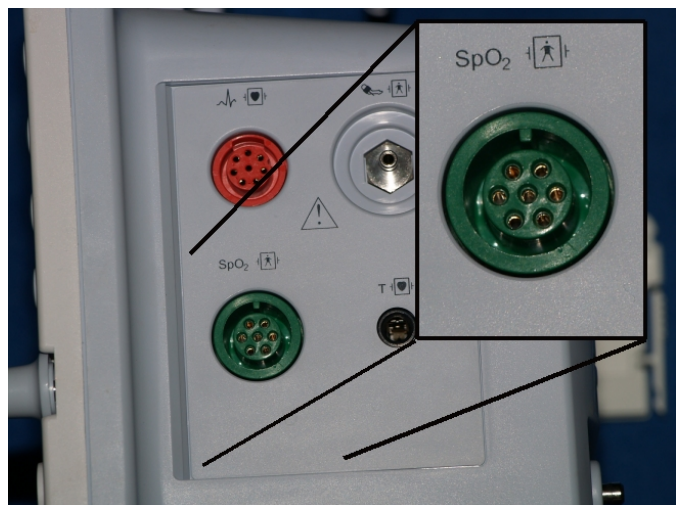

Connector on Monitor

3. No Post Style : There is one other type possible, though it is rare. It looks the same as the new style, but lacks a center post in the connector.

Item Code: B-SPCBL-NP, Item Description: SpO2 Extension Cable - No Post

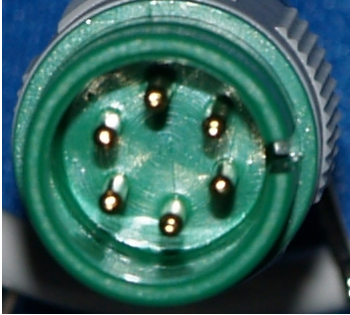

B. 3(or 5) Lead ECG Cable and Extension Cable

We have two variations on the 3(or 5) Lead ECG Cable and Extension Cable(Pictures shown below are 3 Lead ECG Cable and Extension Cable):

1. Old Style

▶ 3 Lead ECG Cable and Extension Cable : Pictured

Item Code: B-WIRE-O, Item Description: 3 Lead ECG Cable (Alligator Type) - Old

Item Code: B-CBL-O, Item Description: 3 Lead ECG Extension Cable - Old

▶ 5 Lead ECG Cable and Extension Cable : Not Pictured

Item Code: B-WIRE5-O, Item Description: 5 Lead ECG Cable (Alligator Type) - Old

Item Code: B-CBL5-O, Item Description: 5 Lead ECG Extension Cable - Old



2. New Style

▶ 3 Lead ECG Cables and Extension Cable : Pictured


Item Code: B-WIRE-N, Item Description: 3 Lead ECG Cable (Alligator Type) - New

Item Code: B-CBL-N, Item Description: 3 Lead ECG Extension Cable - New

▶ 5 Lead ECG Cable and Extension Cable : Not Pictured

Item Code: B-WIRE5-N, Item Description: 5 Lead ECG Cable (Alligator Type) - New

Item Code: B-CBL5-N, Item Description: 5 Lead ECG Extension Cable - New

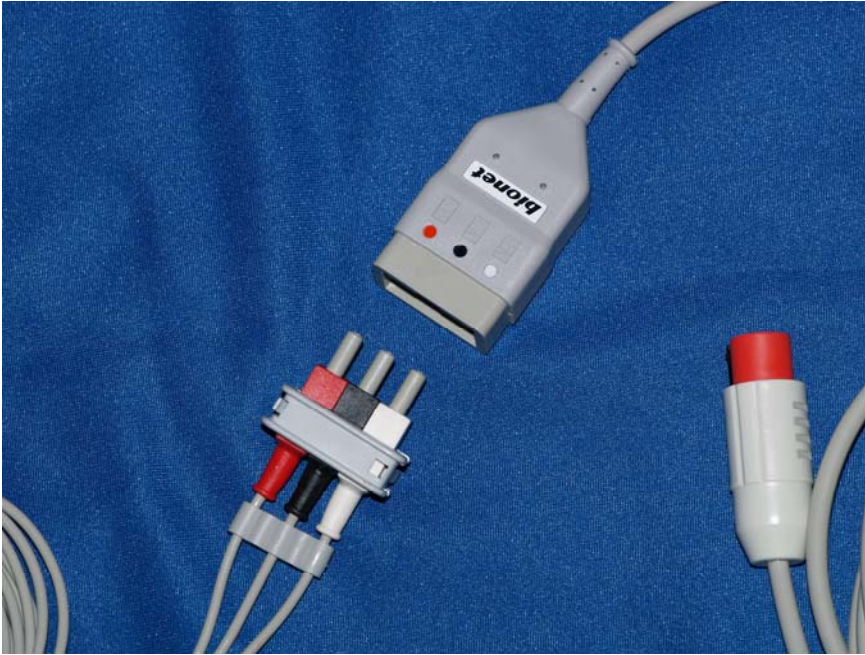

3. BM5Vet All in One Cable

If the BM5Vet has the ECG connector in the Photo on the right, then you need the following ECG cables.



▶ **Three(3) Lead All in One ECG Cable**

Item Code: B-AECG, Item Description: 3 Lead All in One ECG Cable



▶ **Five(5) Lead All in One Cable**

Item Code: B-AECG5 Item Description: 5 Lead All In One ECG Cable



For any questions, please contact Customer Service at 1.877.924.6638 or support@bionetus.com.